**CORPO  
MED**<sup>®</sup>  
Gesundheitskissen

**CorpoMED**<sup>®</sup>  
*Das Original*

Cotton Cover by Öko-Tex Standard 100, washable to 60° C, delicate wash.

CorpoMED does not assume any liability for wrong care with the cushion!

## Wash Instructions

Washing and dry of the cushions only with surely closed outside cover or laundry bag and only to 60°C. Only washing machines use in which the cushion can freely move. The cushion do not press under pressure or tension in the laundry drum. Stamping out the filling can damage her washing machine. CorpoMED takes over no guarantee on damages which are due to stamping out the filling. Only detergents without chlorine use (Dry Cleaner)!

## Attention:

Running out filler can affect the respiratory tract. The cushion may be used only with outside relation. The internal zipper (if available) open only to the fill-up.

## Advice:

Some disinfectants can cause yellow colourings. The change of disinfectants should be avoided, particularly with the use one By means of for a longer period, because otherwise product damages are not to be excluded. We can have on the raw materials and on the production of the disinfectants no influence. In this respect we reject any guarantee achievements which are based on explanations of our suppliers to the "disinfectant permanence"

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**CorpoMED Gesundheitskissen GmbH**